


**90 – Formblatt**  
**Instruction for use**

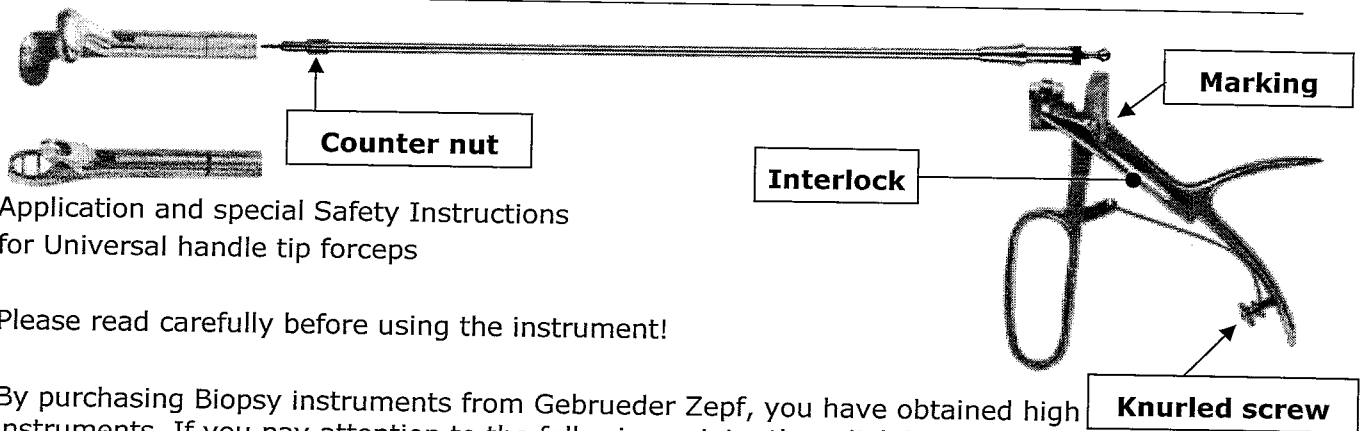

Application and special Safety Instructions  
 for Universal handle tip forceps

Please read carefully before using the instrument!

By purchasing Biopsy instruments from Gebrueder Zepf, you have obtained high instruments. If you pay attention to the following points, the reliability and function will be assured for a long time.

Reprocessing and Maintenance	The reprocessing (sterilization) & maintenance of instruments is not described in this instruction. For these processes, please read our "Instructions for Use - Reprocessing"  The following instruction gives manufacturer's advice for the proper use, secure handling and functional explanations of this instrument.
Before First Use	All instruments delivered by Gebrueder Zepf are non-sterile and must be cleaned and sterilized before first use.


**Assembly of single components**

To assure function for a long time, a marking has been applied to the handle ("Montage") that facilitates assembly.

You only need to:

1. Open interlock.
2. Insert tube into Universal handle and close interlock.
3. Close handle until marking arrows point to each other. In this position, screw closed tip onto tube thread.
4. Fix tip in requested working position by using counter nut.
5. The cutting angle can be adjusted into 8 different positions (45° each) by turning the tube, while interlock is opened.

**Please make sure interlock is closed again before using the instrument.**

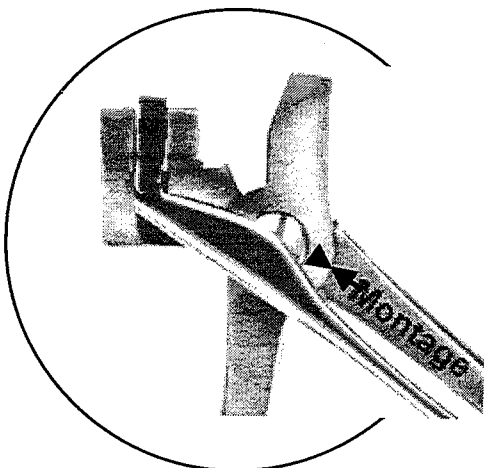

Hints	Instruction
Material that can be cut	Only tissue, cartilage or similar can be cut with forceps.
Overstraining	If the handle is closed with excessive pressure, the string inside the tube can be damaged or even break.

**90 – Formblatt**  
**Instruction for use**



Spring pressure on handle	Spring tension can be adjusted by turning knurled screw on handle. (>>> clockwise = more tension, >>> counter-clockwise = less tension)
Repairs	Faults requiring repair are rare if the guidelines mentioned above are carefully followed. Should you, however, return an instrument requiring repairs, we would kindly ask you to clean and disinfect it before returning.
Additional Information	Guarantee claims are void if it is found that the instruments have been handled incorrectly!
Manufacturer Contact :	(Postal address: see above) Tel.: +49 7424 95 72 0, FAX : +49 7424 95 72 40, email : gebr.zepf@t-online.de