

TAPKERRISON<sup>®</sup>  
take apart





**TAP KERRISON<sup>®</sup>**  
take apart

➤ **EASY TO CLEAN**

Very easy to disassemble and reassemble again in only 3 steps. Easy to clean, easily sterilizable, no debris can hide in shaft.

➤ **NO SPRING SYSTEM**

No problems with springs slipping off, better view during surgery. No risk of glove getting caught in spring.

➤ **COMPACT AND POWERFUL**

Small handle \*, but at the same time 14mm mouth opening. Perfect for surgeons who do not like the large Kerrison handles, but do not want to miss the advantages of a large mouth opening.

➤ **SIMPLE IMPLEMENTATION**

Our tests have shown that personnel, of all profiles and education levels, quickly learn the technique of disassembling and reassembling the TAP Kerrison punches.

\* also available for large handle Kerrisons

Take apart  
in **3** steps

**TAPKERRISON**<sup>®</sup>  
take apart



**1** TAP Kerrison punches can be disassembled in just a few seconds. This new technology will solve all problems of cleaning classic Kerrison punches. The whole procedure is described in the following steps:

**a** On normal position, push the button on Kerrison body, as shown on picture A.

**b** Now the front handle can be pulled down.

**c** Pulling the upper shaft backwards and up, your TAP Kerrison punches are disassembled and ready for cleaning.

\* The Kerrison punches can be reassembled using backward procedure.

TECHNOLOGY. SIMPLICITY. SOLUTION.

WWW.GZ-MEDIZINTECHNIK.COM



Check out our latest products. For more detailed information please contact Gebrueder Zepf headquarters or your local resaler. You can arrange a demo or ask for promotional material, available on DVD, in print and on the internet.


solution

CLICKERRISON<sup>®</sup>  
turnable



simplicity

TAPKERRISON<sup>®</sup>  
take apart

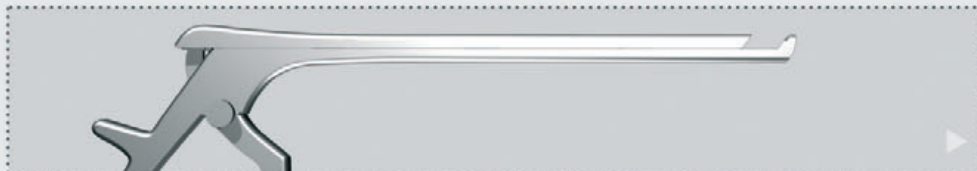

technology

SMARTHANDLE<sup>®</sup>  
color code grip

